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Instruction Manual

Manual de instrucciones

Mode d'emploi

ECO – MATIC®

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with bottle attachment

con adaptador para frascos

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by **HSW**

for veterinary use only

0,3 ml, 0,5 ml, 1 ml, 2 ml or 5 ml

Henke-Sass, Wolf GmbH
Keltenstrasse 1
D-78532 Tuttlingen Germany
Tel. +49 7462 94660
Fax +49 7462 9466 52 08
www.henkesasswolf.de
E-mail: info@henkesasswolf.de

ECO-MATIC®

von HSW

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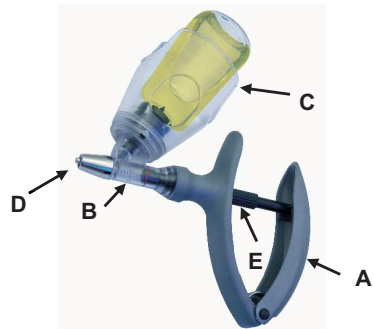
- A) ergonomic designed handle
- B) plastic barrel with bottle attachment
- C) durable plastic basket
- D) metal needle fitting, Luer Lock
- E) dosage setting

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2) CONNECT THE BOTTLE

1. Pierce the rubber cap of the bottle with a sterile needle before you attach the bottle into the spike.
2. Insert the bottle into the basket.
3. Line up the bottle, by using the viewing window of the basket, in the centre of the rubber cap and pierce the spike into the bottle while rotating the bottle slightly.

3) EXCHANGE THE BOTTLE

- Pull out the bottle by using the grip hollow on both sides of the bottle.

4) SET DOSE

- For 0,5ml, 1 ml, 2ml and 5 ml: Turn the dose adjuster either clockwise or counter-clockwise until front of the piston is aligned with the dose setting printed on the barrel.
For 0,3 ml: Turn the dose adjuster either clockwise or counter – until the front of the O-ring is aligned with the dose setting printed on the barrel.
- In some syringe models the dosage is fixed at a certain volume and cannot be adjusted manually.

Dose setting 0,5, 1, 2, 5ml

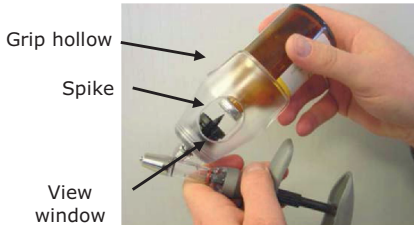
Dose setting 0,3ml

5) **FILLING OF THE HSW ECO-MATIC®**

1. Remove protection cover of the needle.
2. Hold the ECO-MATIC® at a slight incline and take care that the spike is below the liquid to avoid suck in of air into the barrel.
3. Squeeze adjustable handle and then release quickly to draw fluid into the barrel.
4. To remove air bubble inside the barrel, repeat step (3).
5. To fill the syringe and prepare for injection please make sure that the syringe is adjusted on volume, 0,3 ml (for 0,3 ml syringe) 0,5 ml (for 0,5 ml syringe) and at least 0,5 ml (for 1ml syringe) to prime and that no air bubbles are visible. Hold the syringe upright and squeeze the plunger so that air bubbles can be released. From now on any dosage will be possible.

Application with adapter for threaded bottles:

Hold the HSW ECO-MATIC® with the suction tip downwards and screw the threaded bottle into the adapter.

→ continue with step 5.2.

Dismantling is carried out in the reverse order.

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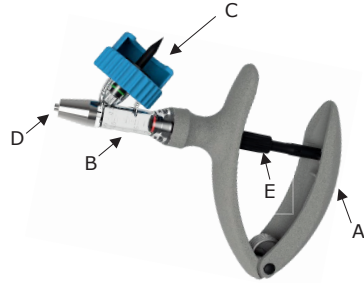
ECO-MATIC®

by HSW

- **Automatic self-filling syringe for injection**
- **with bottle attachment**

- A) ergonomic designed handle
- B) plastic barrel with bottle attachment
- C) universal collar
- D) metal needle fitting, Luer Lock
- E) dosage setting

Instruction for use

The HSW ECO-MATIC® self-filling dosing syringe is designed for continuous use, only the O-ring and the barrel can be changed if necessary.

It is essential that the syringe is cleaned before each use. Please follow the manufacturers recommendations with any cleaning or disinfectant solutions used.

1) TO FIT THE NEEDLE

Attention:

- For your safety: Leave the protective cover (disposable needles) on the needle until ready for use.
- Insert needle onto Luer Lock fitting and secure the needle clockwise.
- Disconnect the needle fitting the opposite way.

2) CONNECT THE BOTTLE

fig.1a:
Select the corresponding adapter for the corresponding bottle.
Introduce bottle in universal

fig.2:
Stick the spike central in the bottle, **without** turning the bottle itself.

fig.3:
Tighten the adapter to the syringe **without** turning the bottle itself.

fig.1b:
FOR THREADED BOTTLES ONLY:
Hold the HSW ECO-MATIC® with the spike downwards and screw the threaded bottle into the adapter. Dismantling is carried out in the reverse order.

Universal adapter set for all common sizes of medicine bottles:

- White UBA for bottle neck size 20 mm
- Blue UBA for bottle neck size 30 mm
- Red UBA for bottle neck size 30.5 mm
- Green UBA for bottle neck size 33 mm
- Black UBA for bottle neck size 34 mm

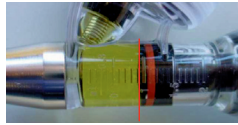
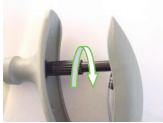
Optional tube adapter (not included) for connecting a medicine bottle via a tube (ID Ø4-6mm).

Tube adapter and tube sold separately.


3) **SET DOSE**

- For 0.5ml, 1 ml, 2ml and 5 ml: Turn the dose adjuster either clockwise or counter-clockwise until front of the piston is aligned with the dose setting printed on the barrel.
For 0.3 ml: Turn the dose adjuster either clockwise or counter – until the front of the O-ring is aligned with the dose setting printed on the barrel.
- In some syringe models the dosage is fixed at a certain volume and cannot be adjusted manually.

Dose setting 0.5, 1, 2, 5ml

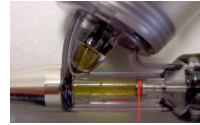Dose setting 0.3ml

4) **FILLING OF THE HSW ECO-MATIC®**

1. Remove protection cover of the needle.
2. Hold the ECO-MATIC® at a slight incline and take care that the spike is below the liquid to avoid suck in of air into the barrel.
3. Squeeze adjustable handle and then release quickly to draw fluid into the barrel.
4. To remove air bubble inside the barrel, repeat step (3).
5. To fill the syringe and prepare for injection please make sure that the syringe is adjusted on volume, 0.3 ml (for 0.3 ml syringe), 0.5 ml (for 0.5 ml syringe) and at least 0.5 ml (for 1ml syringe) to prime and that no air bubbles are visible. Hold the syringe upright and squeeze the plunger so that air bubbles can be released. From now on any dosage will be possible.

INSTRUCTION AFTER USE OF THE HSW ECO-MATIC®

Remove all traces of material from syringe by flushing thoroughly with a hot water detergent mix. Rinse thoroughly with clean, warm water.

Attention!

If you detect that the syringe draws air after the described cleaning, it is possible that a valve is fixed in the front metal approach. The valve can be released by knocking slightly the front of the syringe against an object, e.g., the edge of a table, and afterwards rinsing the syringe with warm water.

DISINFECTION

It is recommended that the HSW ECO-MATIC® and a supply of needles or nozzles be disinfected before and after each use.

1. Suck „hot“ water into the barrel by using an attached bottle with warm water.
2. Suspend the complete HSW ECO-MATIC® including needles or nozzles in a container of water and boil it for at least 20 minutes. Ideally, the cylinder will be disassembled from the handle and the syringe will be boiled in a disassembled condition.
3. Remove the HSW ECO-MATIC® from container, wrap cloth around the handle and pump it dry.

TO LUBRICATE:

Remove plastic barrel from the handle by turning it. Lubricate piston and glass barrel with 100% pure oil, cold squeezed or equivalent vegetable oil.

Please follow the manufacturers durability and storage life instruction.

Do not use mineral oil, silicone, WD-40, glycerine or alcohol. Petroleum based products harden the piston, reducing the life of the component.

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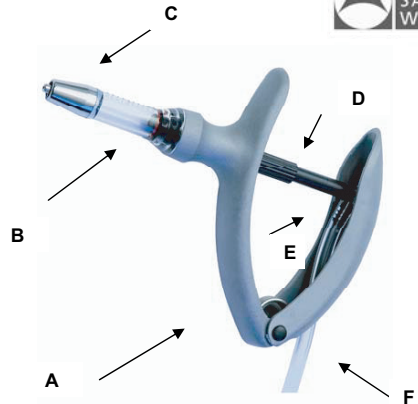
ECO – MATIC®
by **HSW**

- Automatic self-filling syringe for injections
- with tubing attachment

- A) ergonomic designed handle
- B) plastic barrel with piston and O-ring
- C) metal needle fitting - Luer Lock
- D) dosage adjustment
- E) tube inlet socket
- F) tube insert holde in the handle

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It is essential that the syringe is cleaned before each use. Please follow the manufacturers' recommendations with any cleaning or disinfectant solutions used.

1) TO FIT THE NEEDLE

Attention:

- For your safety: Leave the protective cover (only on disposable needles) on the needle until ready for use.
- Insert needle onto Luer Lock fitting and secure the needle clockwise.
- Disconnect the needle twisting the opposite way.

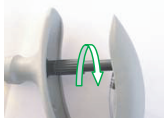
2) ATTACH DRAW – HOSE INTO THE INLET SOCKET

1. Insert draw hose into the tube inlet socket of the syringe.
2. Push the end of the draw hose through tube insert hole on the handle and straighten the tube.

3) SET DOSE

- Turn the dose adjuster either clockwise or counter-clockwise until front of the piston is aligned with the dose setting printed on the barrel.
- In some syringe models the dosage is fixed at a certain volume and cannot be adjusted manually.

Dose setting

4) FILLING OF THE ECO-MATIC®

1. Remove protection cover on the needle.
2. Hold ECO-MATIC® at a slight incline.
3. Squeeze adjustable handle and then release quickly to draw fluid into barrel.
4. To remove air bubble inside barrel, repeat step (c) taking care not to waste fluid.
5. To fill the syringe and prepare for injection please make sure that the syringe is adjusted on volume, 0.5 ml (for the 1 ml fine dosage syringe) to prime and that no air bubbles are visible. Hold the syringe upright and squeeze the plunger so that air bubbles can be released. From now on any dosage will be possible.

INSTRUCTION AFTER USE OF THE ECO-MATIC®

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Attention! If you detect that the syringe draws air after the described cleaning, it is possible that a valve is fixed in the front metal approach. The valve can be released by knocking slightly the front of the syringe against an object, e.g., the edge of a table, and afterwards rinsing the syringe with warm water.

DISINFECTION

It is recommended that the ECO-MATIC® and a supply of needles or nozzles be disinfected before and after each use.

1. Suck "hot" water into the barrel by using an attached bottle with warm water.
2. Suspend the complete ECO-MATIC® including needles or nozzles in a container of water and boil for up to 20 minutes. Ideally, the cylinder will be disassembled from the handle and the syringe will be boiled in a disassembled condition.
3. Remove the ECO-MATIC® from container, wrap cloth around the handle and pump it dry.

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Do not use mineral oil, silicone, WD-40, glycerine or alcohol. Petroleum based products harden the piston O-ring, reducing the life of the component.

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© copyright · Henke-Sass, Wolf GmbH · 700038-09-24 · Änderungen in Technik und Design vorbehalten · Printed in Germany
© copyright · Henke-Sass, Wolf GmbH · 700038-09-24 · Subject to amendments in technology and design · Printed in Germany
© copyright · Henke-Sass, Wolf GmbH · 700038-09-24 · Reservada los derechos de modificaciones técnicas y diseño · Printed in Germany
© copyright · Henke-Sass, Wolf GmbH · 700038-09-24 · Sous réserve de modifications dans le design et la technique · Printed in Germany
© copyright · Henke-Sass, Wolf GmbH · 8300053097- 09-24 · Änderungen in Technik und Design vorbehalten · Printed in Germany
© copyright · Henke-Sass, Wolf GmbH · 8300053097- 09-24 · Subject to amendments in technology and design · Printed in Germany