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Instruction Manual

Manual de instrucciones

Mode d'emploi

UNI - MATIC®

for veterinary use only

by **HSW**

1 / 2 / 5 ml

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
UNI-MATIC®
by **HSW**

- ▶ Automatic self-filling syringe for injections
- ▶ with bottle attachment or tubing

- A) Ergonomic designed handle
- B) Glass barrel
- C) Precise metal piston
- D) Dosage setting
- E) Bottle top for universal adapters / tubing adapter

- 1) Bottle feeding syringe
- 2) Tube feeding syringe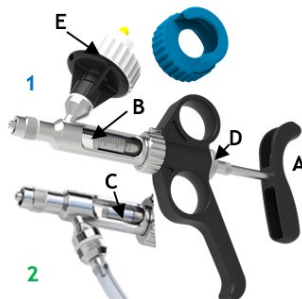

The UNI-MATIC® self-filling dosing syringe is designed for continuous use.

The UNI-MATIC® syringes also feature an innovative handle design that allows for adaptability from a tube feed or a bottle attachment syringe. It is essential that the syringe is cleaned after each use. Please follow the manufacturers recommendations with any cleaning or disinfectant solutions used.


TO FIT THE NEEDLE

Attention:

- For your safety: Leave the protective cover (only on disposable needles) on the needle until ready for use.
- Insert needle on to Luer Lock fitting and secure the needle clockwise.
- Disconnect the needle fitting the opposite way.


1) USE AS TUBE-FED SYRINGE

1. Attach one end of the tube to the syringe.
2. Attach the other end to the ascending tube or draw-off cap when used.
3. Insert the ascending tube into the bottle.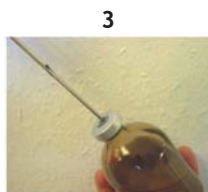

2) USE AS BOTTLE-FED SYRINGE


1. Pierce the rubber cap of the bottle with sterile needle before you attach the bottle into the spike.
2. Follow these steps: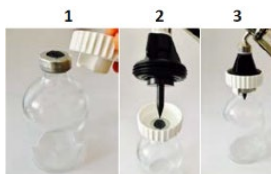

- Insert bottle into universal adapter
- Insert spike into the center of bottle rubber bung
- Screw the bottle with the respective adapter onto the thread until it is firmly seated.

3) SET DOSE

1. Loosen dose adjustment lock nut
2. Rotate dosing adjustment to set the required dosage. Retighten lock nut.

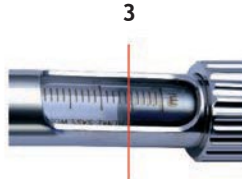Dose setting

4) FILLING OF THE UNI-MATIC®

1. Remove protection cover of the needle.
2. Hold the UNI-MATIC® at a slight incline and take care that the spike is below the liquid to avoid sucking in of air into the barrel.
3. Squeeze adjustable handle and then release quickly to draw fluid into the barrel.
4. To remove air bubble inside the barrel, repeat step 3.

INSTRUCTION AFTER USE OF THE UNI-MATIC®

Remove all traces of material from syringe by flushing thoroughly with a hot water detergent mix.
Rinse thoroughly with clean, warm water.

DISINFECTION / STERILISATION / AUTOCLAVE

It is recommended that the UNI-MATIC® and a supply of needles or nozzles be disinfected before and after each use.

1. Suck "hot" water into the barrel by using an attached bottle with warm water.
2. Suspend the complete UNI-MATIC® including needles or nozzles in a container of water and boil for up to 20 minutes.
3. Remove the UNI-MATIC® from container, wrap cloth around the handle and pump it dry.

Sterilization at 134°C / 273°F

- remove basket, barrel sleeve and glass barrel from handle before autoclaving.

TO LUBRICATE

Dismantle cylinder and lubricate piston with 100% pure olive oil, cold squeezed or equivalent vegetable oil. Please follow the manufactures durability and storage life instruction.

Do not use mineral oil, silicone, WD-40, glycerin or alcohol. Petroleum based products harden the o-rings, reducing the life of the component.

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© copyright · Henke-Sass, Wolf GmbH · 800020740-10/24 · Änderungen in Technik und Design vorbehalten · Printed in Germany
© copyright · Henke-Sass, Wolf GmbH · 800020740-10/24 · Subject to amendments in technology and design · Printed in Germany
© copyright · Henke-Sass, Wolf GmbH · 800020740-10/24 · Reservados los derechos de modificaciones técnicas y de diseño · Impreso en Alemania
© copyright · Henke-Sass, Wolf GmbH · 800020740-10/24 · Sous réserve de modifications dans le design et la technique · Printed in Germany