HSW ECO-MATIC® 6ml / 10ml

with Back Bottle Mount or Tubing

for veterinary use only

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Instruction Manual

Manual de instrucciones

Mode d'emploi

HSW ECO-MATIC® 6ml / 10ml

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
HSW ECO-MATIC® 6ml / 10ml

- *Automatic self-filling syringe in different versions for oral and pour-on use or injection*


Back Bottle Mount

- A) Ergonomic handle
- B) Swivel nut
- C) Cylinder with piston and o-ring
- D) Adjusting wheel for dosage setting
- E) Universal Back Bottle Mount Adapter
- F) Tubing Adapter
- G) Threaded Back Bottle Mount Adapter
- H) Luer Lock Attachment (alternative Drench and Pour-On Nozzles)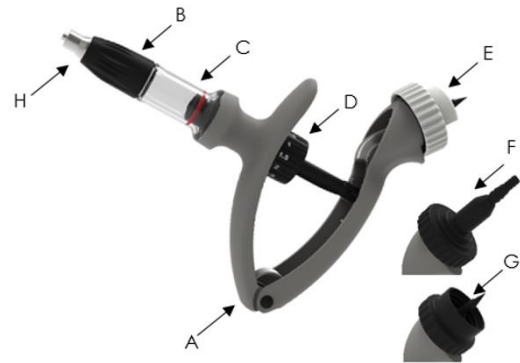

Tubing

- A) Ergonomic handle
- B) Swivel Nut
- C) Cylinder with piston and o-ring
- D) Adjusting wheel for dosage setting
- E) Tubing Adapter
- F) Drench and Pour-On Nozzles (alternative Luer Lock Attachment)

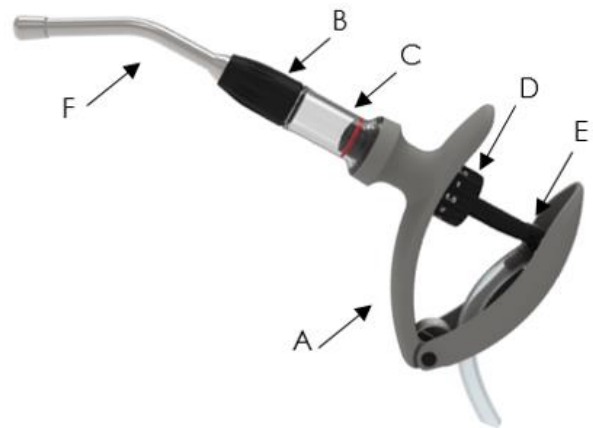Nozzles for HSW ECO-MATIC® 6ml and 10 ml

The applicator has a standardized front part, to which various Nozzles, for Drenching and Pour-On, as well as a Luer Lock attachment for injections, can be attached. (Accessories are available separately).

This uniform front part is used in the majority of HSW applicators (HSW ECO-MATIC® 6ml / 10ml, HSW ECO-MATIC® DRENCHER 12.5ml / 30ml, HSW DRENCH-MATIC®). That enables an interchange of the Nozzles between the applicators.

- a) Drench Nozzle (14 cm)
- b) Drench Nozzle (11 cm)
- c) Pour-On Nozzle
- d) Luer Lock attachment
- e) Nozzle straight, metal (Flat Jet Nozzle)
- f) Nozzle straight, plastic (Flat Jet Nozzle)

User Manual

The HSW ECO-MATIC® 6ml and 10ml were developed for oral and Pour-On application or injection of solutions. The ergonomic, user-friendly design as well as all high-strength materials, that were used in the manufacturing, guarantee fatigue-free use as well as a long service life of your HSW ECO-MATIC®.

When using the HSW ECO-MATIC®, adhere to the respective manufacturer's instructions for the medication!

General warnings:

- ⚠ The HSW ECO-MATIC® is only intended to be used for the treatment of animals. The application on humans may cause damage to their health.
- ⚠ Do not point the HSW ECO-MATIC® towards any part of your body or towards other people! This can lead to injuries or accidental injections.
- ⚠ In case of accidental injection or application of a substance, consult a doctor immediately and present the information sheet of the respective substance to the doctor.
- ⚠ The use of non-approved substances or the overdose / underdose of these approved substances can lead to adverse health effects in animals and/or later in humans.
- ⚠ Caution, parts of the ECO-MATIC® can break / be damaged. Damaged parts can cause injuries to the operator and animals.
- ⚠ Caution, sharp needles and broken needles can cause harm to the operator and animals. Use needles with care and do not bend them.
- ⚠ **Use care around** the spike in the vaccine vial holder area. It may contain residues of substances. In case of a caused spike wound, consult a doctor immediately and present the information sheet of the respective substance to the doctor.

General notes:

- Substances to be administered with the HSW ECO-MATIC® are not provided by HSW. The manufacturer's instructions must be adhered.
- The HSW ECO-MATIC® may only be used by operators who have read and understood the user manual in full.
- It is recommended to wear protective goggles, a mouth guard and rubber gloves during the use and cleaning of the HSW ECO-MATIC®.

1) PREPARATING THE HSW ECO-MATIC® FOR USE

With Drench or Pour-On Nozzle

1. Unscrew the swivel nut and remove the present adapter.
2. Push the Drench or Pour-On Nozzle tube into the cylinder with slight pressure, rotated in the wanted position, and fasten it with the swivel nut.

With Luer Lock attachment for injection needles

1. Unscrew the swivel nut and remove the present adapter.
2. Push the Luer Lock attachment into the cylinder with slight pressure and fasten it with the swivel nut.
3. Insert appropriate needle into Luer Lock attachment.

Caution:

- Please remove the protective cap of the needle (when using disposable needles) only when using the syringe.
- Place the needle on the Luer Lock attachment and fasten it by turning the needle clockwise (Attention: the needle should only be assembled or disassembled without tools).
- For loosening the needle attach the protective cap and turn it the other way round.

Back Bottle Mount

Preparing the HSW ECO-MATIC® with tubing adapter

Fig. 1

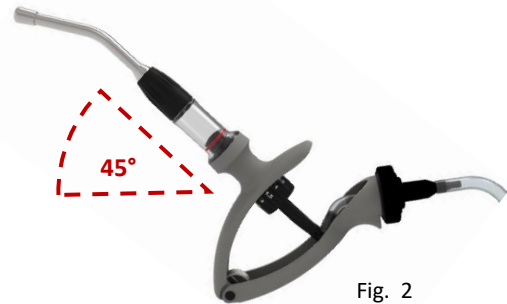
Fig. 2

1. Attach the tube to the tubing adapter (F) of the HSW ECO-MATIC® (Fig. 1).
2. Option 1: Attachment of the tube to the adapter of the back canister. Option 2: Attachment of the tube to the draw-off cap when used with a bottle (Abb. 1).
3. Set the HSW ECO-MATIC® to the largest possible dosage and hold it at an angle of at least 45° pointing upwards (Fig. 2).
4. Compress the handles and let them slide apart again in order to suck liquid into the cylinder.
5. If there are air bubbles inside the cylinder, repeat steps 3 and 4 until no more air bubbles appear.
6. Make sure that the HSW ECO-MATIC® is completely filled and that there are no air bubbles in the cylinder. If necessary, point the tip vertically upwards so that the air can escape while compressing the handles again.
7. After this initial setting, any smaller dosing quantity can be set. (see chapter: Setting the dosing quantity)

Preparing the HSW ECO-MATIC® with back bottle mount adapter

Fig. 3

Fig. 4

Fig. 5

Universal Back Bottle Mount Adapter:

1. Pierce the rubber membrane of the bottle centrally with a cannula to let possible excess pressure escape from the bottle.
2. Select the universal back bottle mount adapter (E) suitable for the corresponding bottle neck size.
3. Insert the head of the bottle sideways into the back bottle mount adapter (Fig. 3).
4. Turn the HSW ECO-MATIC® upside down so that the suction tip points downwards (Fig. 4).
5. Pierce the suction tip centrally into the rubber membrane of the bottle.
6. Screw the adapter onto the HSW ECO-MATIC® without turning the bottle (Fig. 5).
7. Turn the HSW ECO-MATIC® back upside again, set it to the highest possible dosage and hold it at an angle of at least 45° pointing upwards (Fig. 2).

8. Compress the handles, move the syringe into a horizontal position and let the handles slide apart to draw liquid into the cylinder.
9. If there are air bubbles inside the cylinder, repeat steps 7 and 8 until no more air bubbles appear.
10. Make sure that the HSW ECO-MATIC® is completely filled and that there are no air bubbles in the cylinder.
11. After this initial setting, any smaller dosing quantity can be set. (see chapter: Setting the dosing quantity)

Threaded Back Bottle Mount Adapter:

1. Pierce the rubber membrane of the bottle centrally with a cannula to let possible excess pressure escape from the bottle.
2. Screw the threaded back bottle mount adapter (G) onto the thread of the HSW ECO-MATIC®.
3. Turn the HSW ECO-MATIC® upside down so that the suction tip points downwards (Fig. 4).
4. Pierce the suction tip centrally into the rubber membrane of the bottle.
5. Screw the bottle onto the threaded back bottle mount adapter on the HSW ECO-MATIC® (Fig. 5).
6. Turn the HSW ECO-MATIC® back upside again, set it to the highest possible dosage and hold it at an angle of at least 45° pointing upwards (Fig. 2).
7. Compress the handles, move the syringe into a horizontal position and let the handles slide apart to draw liquid into the cylinder.
8. If there are air bubbles inside the cylinder, repeat steps 6 and 7 until no more air bubbles appear.
9. Make sure that the HSW ECO-MATIC® is completely filled and that there are no air bubbles in the cylinder.
10. After this initial setting, any smaller dosing quantity can be set. (see chapter: Setting the dosing quantity)

Tubing

Preparing the HSW ECO-MATIC® with tubing adapter

Fig. 1

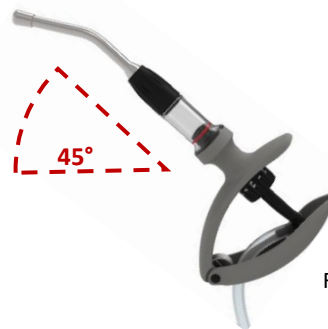
Fig. 2

1. Insert the tube through the inlet hole in the movable handle, from the outside to the inside, and attach the tube to the hose attachment of the HSW ECO-MATIC® (Fig.1).
2. Option 1: Attach the tube to the tubing attachment of a back pack canister.
Option 2: Attach the tube to a draw-off cap when using a bottle.
3. Set the HSW ECO-MATIC® to the highest possible dosage and hold it at an angle of at least 45° upwards (Fig. 2).
4. Compress the handles and let them slide apart again in order to suck liquid into the cylinder.
5. If there are air bubbles inside the cylinder, repeat steps 3 and 4 until no more air bubbles appear.
6. Make sure that the HSW ECO-MATIC® is completely filled and that there are no air bubbles in the cylinder. If necessary, point the tip vertically upwards so that the air can escape while compressing the handles again.

7. After this initial setting, any smaller dosing quantity can be set. (see chapter: Setting the dosing quantity)

Setting the dosing quantity

Look at the dosing wheel from above and turn it until the arrow embossed in the handle points to the desired dose (selected dose points upwards). The handle can be squeezed less or more according to the setting (maximum distance at maximum dose, minimum distance at minimum dose). A new dose is automatically drawn into the cylinder when letting the handle slide backwards. Do not compress the handles while setting the desired dose. Due to the special mechanism of the dosage wheel, no substance inside the applicator will be wasted during the dosage setting.

2) CLEANING AND REPLACING THE VALVES

When cleaning or replacing the springs, ensure that the cylinder head spring and the piston spring are not interchanged, as they have different spring tensions. The stronger spring is for the cylinder head (spring with fewer windings and valve with a cross at the end) (fig. 6) and the weaker spring (spring with more windings and slot at the end) (Fig. 7) for the piston.

Fig. 6

Fig. 7

Cleaning / replacing the valve on the cylinder head

Fig. 8

1. Unscrew the swivel nut.
2. Remove accessories such as the attachment tube or the Luer Lock attachment.
3. Pull the valve incl. spring and O-ring out forwards against a slight resistance (if necessary with the aid of small pliers).
Do not pull or bend the spring!
4. Oil the cleaned or new valve incl. spring and O-ring and then insert it into the cylinder and let it click into place against a small resistance.
Make sure that the spring and the valve are correctly aligned. (Fig. 8)

Cleaning / replacing the valve in the piston

Fig. 9

Fig. 10

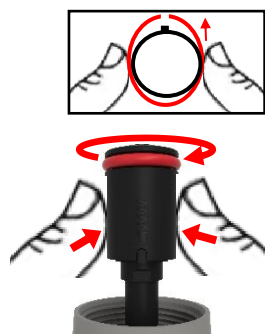
Fig. 11

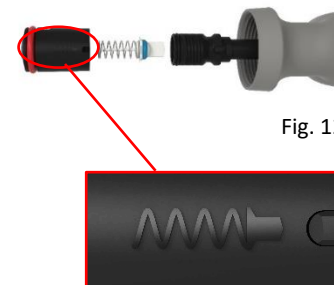
Fig. 12

Fig. 13

1. Unscrew the cylinder from the handle by turning it anticlockwise (Fig. 9).
Caution: Before unscrewing, expel any remaining liquid from the cylinder.
2. Compress the handle of the HSW ECO-MATIC® as if to administer a dose, so that the piston can be easily reached with the fingers (Fig. 10).
3. Squeeze the piston with two fingers on both sides so that the piston deforms oval and thus lifts over the safety lug. Turn the piston anticlockwise and unscrew it (Fig. 11).
Caution: The valve and spring may fall out after unscrewing the piston (Fig. 12).
4. Remove the valve and clean it or replace it with a new valve.
5. Oil the valve including the spring and O-ring, then insert it into the piston rod.
Caution: The correct order and direction in which the valve and spring are inserted into the piston rod must be respected!
For this purpose, there is an embossed directional instruction on the piston (Fig. 13).

6. Screw on the piston and rotate it over a small resistance until the safety lug locks into place.
7. Release the handle and then screw on the cylinder.

3) DISINFECTION & CARE AFTER USE OF THE HSW ECO-MATIC®

After use, remove all residues with the usual cleaning solutions and rinse the HSW ECO-MATIC® thoroughly with warm water.

DISINFECTION

It is recommended to disinfect the HSW ECO-MATIC® and its accessories before and after each application.

- Rinse the HSW ECO-MATIC® with hot water.
- Place the complete HSW ECO-MATIC® with unscrewed cylinder and disassembled accessories in a container with water and boil all components for approx. 20 minutes.
- Option: For thorough cleaning, it is recommended to also remove the valves and boil them separately. (see chapter 4: CLEANING AND REPLACING THE VALVES)
- Remove the HSW ECO-MATIC® after boiling, dry it with a cleaning cloth and allow the disassembled components to air dry.

OILING THE PISTON:

Lubricate the O-ring with medical white oil or alternatively 100% pure extra virgin olive oil, cold pressed or equivalent cooking oil. Observe the manufacturer's shelf life and storage instructions. Do not use mineral oils, silicone oils, WD-40, glycerine or alcohol. Petroleum-based oils can damage the piston or reduce the service life of the product. After oiling, the HSW ECO-MATIC® can be assembled. (screw the cylinder onto the handle)

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© copyright · Henke-Sass, Wolf GmbH · 700038-07/20 · Änderungen in Technik und Design vorbehalten · Printed in Germany
© copyright · Henke-Sass, Wolf GmbH · 700038-07/20 · Subject to amendments in technology and design · Printed in Germany
© copyright · Henke-Sass, Wolf GmbH · 700038-07/20 · Reservado los derechos de modificaciones técnicas y diseño · Printed in Germany
© copyright · Henke-Sass, Wolf GmbH · 700038-07/20 · Sous réserve de modifications dans le design et la technique · Printed in Germany
Henke-Sass, Wolf GmbH