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*Instruction Manual*

*Manual de instrucciones*

*Mode d'emploi*

**FERRO – MATIC®**

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**FERRO-MATIC®**  
by HSW

- *Automatic self-filling syringe for injections*
- *with bottle attachment*

- A) ergonomic designed handle
- B) glass barrel
- C) dosage setting
- D) Universal bottle attachment
- E) Universal bottle collar 20mm
- F) Universal bottle collar 30mm

The FERRO-MATIC® self-filling dosing syringe is designed for continuous use. The syringe can be disassembled very easy for cleaning.

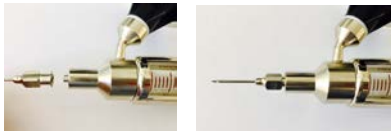
It is essential that the syringe is cleaned before each use. Please follow the manufacturers' recommendations with any cleaning or disinfectant solutions used.

## 1) TO FIT THE NEEDLE

### Attention:

For your safety: Leave the protective cover (only on disposable needles) on the needle until ready for use.

- Insert needle onto Luer Lock fitting and secure the needle clockwise.
- Disconnect the needle fitting the opposite way.

## 2) CONNECT THE BOTTLE

Pierce the rubber cap of the bottle with a sterile needle before you attach the bottle. Choose fitting universal bottle adapter. White for bottle neck  $\varnothing$ 20mm, blue for  $\varnothing$ 30mm.

1. Insert bottle into the universal collar.
2. Insert spike centric into the bottle.
3. Screw the bottle with collar onto the thread until fixed.

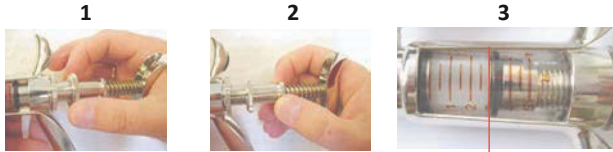
### 3) EXCHANGE THE BOTTLE

Pull out the bottle the opposite way.

### 4) SET DOSAGE

1. Loosen dose adjustment lock nut.
2. Rotate dosing adjustment to set the required dosage. Retighten lock nut.

Dose setting

### 5) FILLING OF THE FERRO-MATIC®

1. Remove protection cover of the needle.
2. Hold the FERRO-MATIC at a slight incline and make sure that the spike is below the liquid to avoid sucking in of air into the barrel.
3. Prime the syringe by pressing the piston rod handle until glass barrel is full of vaccine and free of air bubbles.
4. To remove air bubbles inside the barrel, repeat step.

### 6) INSTRUCTION AFTER USE OF THE FERRO-MATIC®

Remove all traces of material from syringe by flushing thoroughly with a hot water detergent mix. Rinse thoroughly with clean, hot water.

### 7) DISINFECTION

It is recommended that the FERRO-MATIC® and a supply of needles or nozzles be disinfected before and after each use.

1. Suck hot water into the barrel by using an attached bottle with hot water.
2. Suspend the complete FERRO-MATIC® including needles or nozzles in a container of water and boil for up to 20 minutes. Ideally, the cylinder will be disassembled from the handle and the syringe will be boiled in a disassembled condition.
3. Remove the FERRO-MATIC® from container, wrap cloth around the handle and pump it dry.

### 8) TO LUBRICATE

Dismantle the piston and remove glass barrel. Lubricate piston and glass barrel with 100% pure olive oil, cold squeezed or equivalent vegetable oil. Please follow the manufacturer's durability and storage life instruction. Do not use mineral oil, silicone, WD-40, glycerine or alcohol. Petroleum based products harden the piston, reducing the life of the component.

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© copyright · Henke-Sass, Wolf GmbH · 700112- 02/24 · Änderungen in Technik und Design vorbehalten · Printed in Germany
© copyright · Henke-Sass, Wolf GmbH · 700112- 02/24 · Subject to amendments in technology and design · Printed in Germany
© copyright · Henke-Sass, Wolf GmbH · 700112- 02/24 · Reservado a los derechos de modificaciones técnicas y diseño · Printed in Germany
© copyright · Henke-Sass, Wolf GmbH · 700112- 02/24 · Sous réserve de modifications dans le design et la technique · Printed in Germany