-  *Bedienungsanleitung*
-  *Instruction Manual*
-  *Manual de instrucciones*
-  *Mode d'emploi*

ROUX-REVOLVER®

for veterinary use only

by HSW

10 / 30 / 50 ml

*Henke-Sass, Wolf GmbH
Keltenstrasse 1
D-78532 Tuttlingen
Germany
Tel. +49 74 62 94 66 207
Fax +49 74 62 94 66 52 08
www.henkesasswolf.de
E-mail: info@henkesasswolf.de*

ROUX-REVOLVER®

von HSW

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ROUX-REVOLVER®

by HSW

- Dosage syringe for multiple dosing of live stock
- Metal-glass version
- Easy dosage setting
- A design which has been proven for decades

A) Handle

B) Metal tube

C) Glass barrel

D) Dosage setting device

E) Tension knob

Instruction for 10 / 30 / 50 ml

The ROUX-REVOLVER® dosage syringe has been developed for continuous and robust use. The syringe can be disassembled very easy for cleaning.

It is essential that the syringe is cleaned and disinfected before each use. Please follow the manufacturers recommendations with any cleaning or disinfectant solutions used.

TO FIT THE NEEDLE

Attention:

For your safety: Leave the protective cover (only on disposable needles) on the needle until the syringe is ready for use.

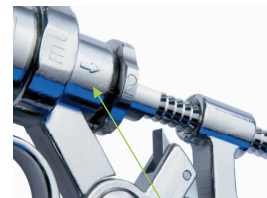
- Insert needle onto Luer-Lock fitting and secure the needle clockwise
- disconnect the needle fitting the opposite way

1) DRAW Medicament

- 1) Press handle (A) altogether
- 2) Turn tension knob (E) clockwise to adjust and tighten the piston depending on the viscosity of the medicament
- 3) Pull the piston rod backwards and suck the medicament in the barrel
- 4) Adjust the tension knob in case the piston needs to be adjusted again
- 5) The syringe is now ready for use

Adjustable piston

Dosage setting device

2) Set dose

The following standard dose settings by turning the dosage setting device (D) are possible

- dosage in 1 ml steps – 1 – 2 – 3 – 4 – 5 ml for 30 and 50 ml
- dosage in 0,5 ml steps – 0,5 – 1 – 1,5 – 2 – 2,5 ml for 30 ml
- dosage in steps 0,25 - 0,5 - 1 ml for 10ml

- special dosage steps for other models are possible

INSTRUCTION AFTER USE OF THE ROUX-REVOLVER®

Remove all traces of material from syringe by flushing thoroughly with hot water detergent mix. Rinse thoroughly with clean, warm water.

Disinfection

It is recommended that the ROUX-REVOLVER® and a supply of needles or nozzles to be disinfected before and after each use.

- 1) Clean the syringe under hot water
- 2) Suspend the complete ROUX-REVOLVER® including needles in a container of water and boil the same for at least 20 minutes
- 3) Remove then syringe from the container, wrap cloth around the handle and pump it dry

TO LUBRICATE:

Remove metal tube from the handle by turning it. Lubricate piston and glass barrel with 100% pure oil, cold squeezed or equivalent vegetable oil.

Please follow the manufacturers durability and storage life instruction.

Do not use mineral oil, silicone, WD-40, glycerine or alcohol. Petroleum based products harden the piston, reducing the life of the component.

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© copyright · Henke-Sass, Wolf GmbH · 8300006358-03/25 · Änderungen in Technik und Design vorbehalten · Printed in Germany
© copyright · Henke-Sass, Wolf GmbH · 8300006358-03/25 · Subject to amendments in technology and design · Printed in Germany
© copyright · Henke-Sass, Wolf GmbH · 8300006358-03/25 · Reservado los derechos de modificaciones técnicas y de diseño · Impreso en Alemania
© copyright · Henke-Sass, Wolf GmbH · 8300006358-03/25 · Sous réserve de modifications dans le design et la technique · Printed in Germany